

IMPLANT COMPATIBILITY CAUTION

Hip Systems:

STEMS, FEMORAL HEADS AND DISTAL CENTRALISERS ARE UNIQUE TO EACH SYSTEM.

- Stems and distal centralisers cannot be mixed and matched between our other hip systems or any other manufacturer's systems. (e.g. Edinburgh™ with Edinburgh™ and Classic Marathon™ with Classic Marathon™ etc.)
- Stems and femoral heads are unique to the extent of:
 - Where stem trunnions are either 10/12 or 12/14, they can be used with a modular head with a 10/12 or 12/14 cavity.
 - Raw Material
 - i) Stainless steel stems can only be used with our stainless steel heads or ceramic heads.
 - ii) Cobalt chrome stems can only be used with our cobalt chrome heads, titanium heads or ceramic heads.
 - iii) Titanium stems can only be used with our cobalt chrome heads, ceramic heads or titanium heads.
- Under no circumstances are our stems to be used with another manufacturer's femoral heads.
- If the indication demands the use of a hybrid 2-piece acetabular cup system, then our stem/head sets may be used with any of our 2-piece systems. Under special circumstances, our stem and distal centralisers and head sets may be matched with another manufacturer's 2-piece acetabular system where our 2-piece systems are not considered appropriate e.g. reconstructive revision acetabular systems. However, in these circumstances, please ensure that our 22.225mm Ø, 26mm Ø, 28mm Ø and 32mm Ø, or whichever diameter chosen, is matched with equivalent a cup of and matching cavity.

Knee Systems

Our femoral, tibial tray, tibial insert and patella button are absolutely unique to our systems. Under no conditions should these be matched with another manufacturer's component.

Extremity Systems

All our lower and upper extremity systems are absolutely unique to our systems. Under no conditions should these be matched with another manufacturer's components.

Implants International Ltd, 71 Jay Avenue, Teesside Industrial Estate, Thornaby,
Cleveland, TS17 9LZ United Kingdom

Tel: +44 (0)1642 769080 Fax: +44 (0)1642 765848

e-mail: enquiries@implantsinternational.com
www.implantsinternational.com

