

PACKAGE INSERT FOR INSTRUMENTATION

Product Specifications

THESE INSTRUCTIONS FOR USE RELATE TO: GENERAL CLASS 1 AND CLASS 11A UNSTERILE INSTRUMENTS USED IN ARTHROPLASTY AND TRAUMA AND SPINAL PROCEDURES.

Important Information For The User

This product is designed to be sterilised. Implants International's instruments are manufactured from high quality surgical stainless steel and co. polymer in accordance with national and international standards. Some instruments are made partly from synthetic materials or aluminium. Instruments should only be used within the framework of the intended function. The connection of instruments and active appliances (e.g. drill and drilling machine) must be carried out in such a way that they cannot work loose. When using screwdrivers, care should be taken with correct sizing and safe connection of screw and instrument.

Cleaning Instruments

Alkaline cleaning solutions should be used for cleaning. Kindly use cleaning solutions that do not corrode aluminium, implant steel and synthetic materials. Implants International Ltd. have constructed these instruments in such a way that they can be dismantled for easy cleaning where necessary.

Side Effects

There are no known side effects and interactions of Implants International's instruments with drugs.

Compatibility

Implants International's instruments are designed to be used with our own implants/instruments.

Sterilization

The following sterilization procedures are permitted by the manufacturer:

Sterilization in steam autoclave.

Caution: Under no circumstances should instruments with synthetic material components be sterilized with dry heat.

CAUTION:

United States Federal Law restricts this device to sale by or on the order of a licensed physician.

Implants International Ltd, 71 Jay Avenue, Teesside Industrial Estate, Thornaby, Cleveland, TS17 9LZ United Kingdom

Tel: +44 (0)1642 769080 Fax: +44 (0)1642 765848

e-mail: enquiries@implantsinternational.com

www.implantsinternational.com

